

M7 BED AND STRETCHER CLEANING INSTRUCTIONS

CAUTION

Before cleaning:

- Unplug the power cord from the mains supply
- Depress the castor pedals into the Brake position

During cleaning:

- Do not use abrasive cleaning products
- Do not use a water temperature of more than 50°C
- Do not use a steam cleaner, washing tunnel, high-pressure spray or hose
- Do not immerse the product in water
- Do not clean or degrease the steel shafts of the actuators
- Use only approved cleaning products. If bleach is necessary strictly adhere to the BLEACH CLEANING instructions
- Keep sharp objects such as razors etc away from the mattress cover
- Do not use oil based products
- Do not unplug the handset or actuators from the control box for cleaning

ALL SURFACES

To wash:

- Wipe with a sponge or soft cloth wetted with warm water containing soap or mild detergent
- Cover can be machine washed in warm water (only bielastic mattress covers)

To dry:

- Wipe with a dry sponge or soft cloth
- Place mattress cover in a dry environment (if further drying is required)
- Cover can be tumble dried at a temperature below 50°C but must be removed from the drier immediately when dry and not be left in the bowl. Interrupting the drying cycle occasionally to redistribute the covers may reduce drying time (only bielastic mattress covers)

To disinfect:

- Wash the surface
- Wipe surface with a sponge or soft cloth wetted with an approved cleaning product
- Wipe off residue with water using a soft cloth or sponge
- Dry

REMEMBER

If a washing machine is used, interrupt the washing cycle occasionally to redistribute the covers. This will ensure that they are properly wetted.

Dry cleaning is not recommended.

MATTRESS FOAM

The Prema Advanced III mattress cover is waterproof and vapor permeable and it features welded seams and a waterproof zip. If the mattress cover is punctured it should be replaced to avoid contamination of the mattress foam.

The mattress foam is not suitable for cleaning.

BLEACH CLEANING INSTRUCTIONS

If none of the approved cleaning products provide the disinfectant level required then use a solution of 5.25% Sodium Hypochlorite diluted to less than 1 part bleach to 100 parts water (volume to volume).

Bleach products are corrosive and degrading in nature and may cause damage to your product. If bleach is used, measures must be taken to ensure the product is rinsed thoroughly with clean water and thoroughly dried. Failure to rinse and dry the product will leave a corrosive residue on the surface possibly causing corrosion.

Failure to follow the above directions may void the product's warranty.

Paraité Road, Bell Block
PO Box 3003, Fitzroy
New Plymouth 4341
New Zealand
T. +64 (6) 755 0976
F. +64 (6) 755 0908

U1/2 Production Road
PO Box 2786, Taren Point
New South Wales 2229
Australia
T. +61 (2) 9540 5118
F. +61 (2) 9540 4840