

M7 EXAMINATION COUCH CLEANING INSTRUCTIONS

CAUTION

Before cleaning:

- Unplug the power cord from the mains supply
- Depress the castor pedals into the Brake position

During cleaning:

- Do not use abrasive cleaning products
- Do not use a water temperature of more than 50°C
- Do not use a steam cleaner, washing tunnel, high-pressure spray or hose
- Do not immerse the product in water
- Do not clean or degrease the steel shafts of the actuators
- Use only approved cleaning products. If bleach is necessary strictly adhere to the BLEACH CLEANING instructions
- Keep sharp objects such as razors etc away from the upholstered top
- Do not use oil based products
- Do not unplug the handset or actuators from the control box for cleaning

ALL SURFACES

To wash:

- Wipe with a sponge or soft cloth wetted with warm water containing soap or mild detergent

To dry:

- Wipe with a dry sponge or soft cloth

To disinfect:

- Wash the surface
- Wipe surface with a sponge or soft cloth wetted with an approved cleaning product
- Wipe off residue with water using a soft cloth or sponge
- Dry

BLEACH CLEANING INSTRUCTIONS

If none of the approved cleaning products provide the disinfectant level required then use a solution of 5.25% Sodium Hypochlorite diluted to less than 1 part bleach to 100 parts water (volume to volume).

Bleach products are corrosive and degrading in nature and may cause damage to your product. If bleach is used, measures must be taken to ensure the product is rinsed thoroughly with clean water and thoroughly dried. Failure to rinse and dry the product will leave a corrosive residue on the surface possibly causing corrosion.

Failure to follow the above directions may void the product's warranty.

Paraitē Road, Bell Block
PO Box 3003, Fitzroy
New Plymouth 4341
New Zealand
T. +64 (6) 755 0976
F. +64 (6) 755 0908

U1/2 Production Road
PO Box 2786, Taren Point
New South Wales 2229
Australia
T. +61 (2) 9540 5118
F. +61 (2) 9540 4840